


GMC+med
www.gmcmed.co.kr

www.gmcmed.co.kr
info@gmcmed.co.kr
+82 10 7580 5542
A-1810, 22, Neuti-ro, Bundang-gu,
Seongnam-si, Gyeonggi-do, Korea



The beauty, Now

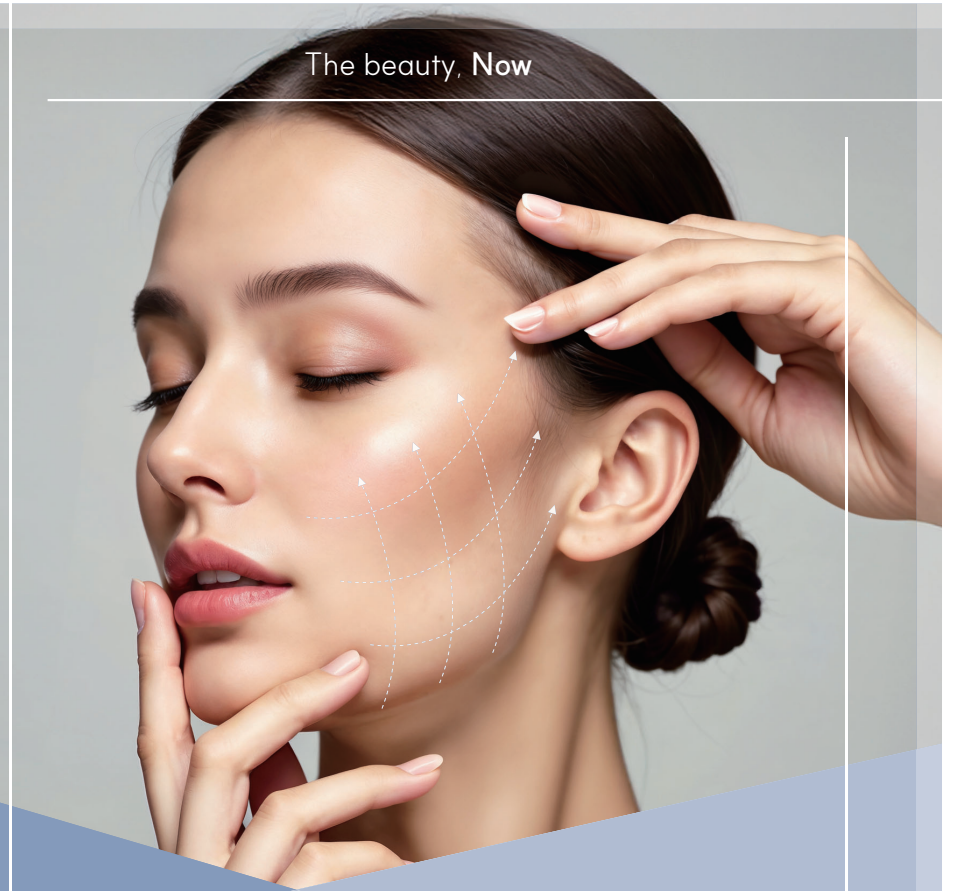

GMC+med

Global-Esthetic Medical Company

CRYSTALDew | **HARMONI** | LuxeContour | BrightSilk PN

www.gmcmed.co.kr

CrystalDew DUAL HA Skin booster

"Simple, painless, and safe with fast absorption and skin benefits."



* Packing unit(1box): 2.5ml/syr

Major ingredients

45mg H-HA (1,100~1,400kDa) + 45mg L-HA (100~300kDa)

CrystalDew DUAL HA Effect



Skin hydration



Skin vivid



Skin elastic & lifting



Skin texture & tone

What is CrystalDew DUAL HA?

1. CrystalDew DUAL HA skin booster, a combination of 90mg of high and low molecular weight Hyaluronic Acid, is a powerful HA revitalizer.
2. Dual (high and low molecular weight) Hyaluronic Acid by thermal hybrid complex process, and it helps for skin reconstruction & remodeling

How to Use & Protocol

1. 5points direct injection each face, or 10points neck 0.2~0.25dosage per one point to deep dermis or subcutaneous layer
2. MTS Device can be used
3. 3~4weeks interval / 3 sessions are recommended



* 5point FAP treatment

What is 5points? It called *Face Aesthetic Points (FAP) and the points are:

1. Zygoma 2. Nasolabial Fold 3. Preauricular 4. Chin 5. Mandibular Angle

What is the benefit 5point FAP treatment by CrystalDew?

The Face Aesthetic Points (FAP) are located in areas devoid of blood vessels or nerve branches, minimizing the risk of bruising and making them suitable for widespread distribution across the face. Therefore, this method ensures safety in injections

CrystalDew NUCLEOTIDE Skin booster

"Recover your damaged tissue to original new cell function"




* Packing unit(1box): 2.0ml/syr

Major ingredients

PN(Polynucleotide) and PDRN(Poly deoxyribonucleotide) from salmon sperm

CrystalDew NUCLEOTIDE Effect

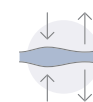

Repairing damaged tissue



Skin brightening




Skin lifting & tightening



Improving wrinkles

What is CrystalDew NUCLEOTIDE?

It contains 2.5% of PDRN & PN complex and providing the nucleotides necessary for Cell or Tissue Regeneration which is involved of anti inflammatory, cell proliferation(Salvage pathway rapid DNA synthesis), and simulates growth factors, and supports tissue remodeling by forming new blood vessels.

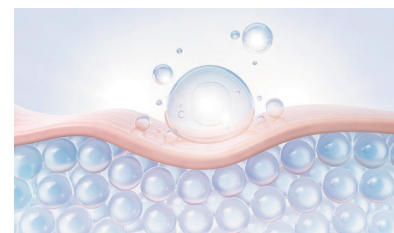

How to Use & Protocol

1. Direct injection each point 0.02~0.03cc drop / injection interval 1 or 2cm
2. 5point direct injection each face, or 10points neck 0.2cc dosage per one point to subcutaneous layer
3. Treatment time: 3~4weeks interval / 3 sessions are recommended



What is PDRN & PN?

Polydeoxyribonucleotide is nature derived regeneration material. It is extracted & purified from Sea trout sperm.

PDRN(Polydeoxyribonucleotide)is optimized low molecule weight DNA, and PN(Polynucleotide) is optimized high molecule weight DNA are premium bioactive ingredients that is synergistically to induce cell differentiation & tissue regeneration, and able to stimulate VEGF(Vascular Endothelial Growth Factors) and Adenosine A2 receptor in the body that enhance microcirculation, stimulate the formation of new blood vessels and help to reduce inflammation for faster healthier skin recovery.

HARMONI PLLA Collagen stimulator

"Restore natural volume and achieve a harmonious look."



* Packing unit(1box): 214mg/vial



* Packing unit(1box): 500mg/vial

Major ingredients

1. HARMONI-214: PLLA 150mg + H-HA & L-HA 64mg
2. HARMONI-500: PLLA 350mg + H-HA & L-HA 150mg

What is HARMONI?

HARMONI is a PLLA[Poly-L-lactic acid] collagen stimulator to induce self collagen production and it acts as a stimulant without generating foreign substances inside of the skin, promoting to production of new collagen and tissue.(Neocollagenesis) By injection this collagen stimulator, it helps restore natural skin volume.

How to Use & Protocol

Face(HARMONI-214) mix with 7cc saline / Body(HARMONI-500) mix with 23cc saline. Keep in room temperature to dilute solution about 3~5hrs

* lidocaine face 1~2cc / body 4~4.5cc just before treatment

1. **Cannula treatment**
Use 25~26G 50~70mm depending on treatment area
2. **Direct injection** : 29G~30G 4~6mm depth to upper subcutaneous
4weeks interval / 2 sessions are needed

What is special?

1. **Dual action for visible skin volume & renewal** – Dual HA (L-HA & H-HA) combined with PLLA delivers immediate hydration and long-lasting regeneration. While hyaluronic acids provide instant moisture, PLLA stimulates fibroblasts, keratinocytes, and adipocytes to boost collagen and elastin production — enhancing skin volume, firmness, and elasticity.
2. **Smart particle technology** – Microsphere particles are specially processed to minimize inflammation and promote safe biodegradation. This reduces the risk of nodules and supports effective cell differentiation at the M2 phase, helping skin regenerate in a healthier, more controlled way.

Effect



Recover
nature volume



Wrinkle and
elasticity
improvement



Collagen
stimulation



Sunken area
improvement

HARMONI

LuxeContour Body & Face volumizer

"Confidence Begins, Contour line what always dreamed of"



Multi-use for Face and Body
* Packing unit(1box): 60mL/vial

Major ingredients

Cross-linked Sodium Hyaluronate 24mg/ml

LuxeContour 4-in-1 Technology



Dense gel



Strong
cohesiveness



Optimal
hardness



Easy molding

What is LuxeContour body & face volumizer?

HA 24mg Cross-linked Body filler (60cc) – delivers superior firmness and elasticity, helping you achieve the ideal shape with long-lasting volume and contour.

Made with Hyaluronic acid with safe raw material which is natural polymer that is characterized by excellent moisture retention and plays import role in maintaining skin structure, and nature moisturizer absorb 1000times the amount of water it weights.

The HA Body 60CC application is used for big volume such as breast and buttocks.

When injected into the body, crossed linked HA attracts moisture in the body and hold the form of gel with maintaining shape and gradually decomposes naturally over time.

What is special?

High-Density HA Body Filler for Sculpted Perfection

01 Volumizing & Contouring



Adds volume, shapes curves, and enhances breasts and buttocks — a non-surgical solution.

02 Safety



Advanced purification removes BDDE, heavy metals, and endotoxins

03 Precision Control



Fine particle size allows easy injection with minimal pressure.

04 Lasting Results



High HA concentration and cross-linking ensure long-lasting effects.

LuxeContour

BrightSilk PN Skinbooster

"Wake Up to Bright, revitalize your skin everyday"




* Packing unit(1box): 2Vials/1Set

Composition

1. Lyophilized Powder: 150 mg / 0.005 oz. × 1 vial
2. Sterilized Liquid: 5 ml / 0.17 fl. oz. × 1 vial

BrightSilk PN Skinbooster Effect

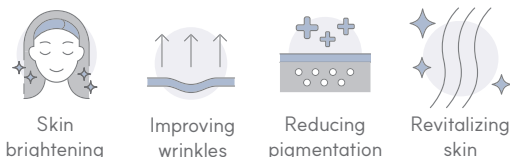

Skin
brightening

Improving
wrinkles

Reducing
pigmentation

Revitalizing
skin

Major ingredients



01. PDRN & PN Complex
Boosts self-regeneration and accelerates wound healing.




02. Glutathione
A powerful antioxidant that protects cells from free radicals and oxidative stress.



03. Tranexamic Acid
Reduces melasma by blocking melanocyte activity and calming inflammation.



04. Hyaluronic Acid
Deep hydration and skin repair with dual molecular weights.

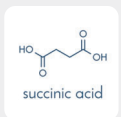

05. Succinic Acid
Anti-inflammatory, antioxidant, moisturizes, and brightens pigmentation.

What is BrightSilk PN skinbooster?

BrightSilk PN Skinbooster is PDRN & PN combination skin regeneration and overall skin rejuvenate mesotherapy product, improve moisture, wrinkles, and solve melanin & pigmentation

Application

Whole face, neck, hand, décolletage

How to Use & Protocol

MTS device, AMTS,
MTS roller (0.2~0.5mm depth)
Derma Shine / Ultrasound device
Ion : No (+)(-) ionization. Use with neutrality



*Suitable for all types of Mesotherapy treatment methods.

1. Treatment time: 2~3weeks interval / 3 sessions are recommended
2. Suitable for mixing other solution like Exosomes, Vitamix C, or Botox

BrightSilk Repair Serum & Cream

"Achieve perfection with the final touch"



* Packing unit(1box): 50ml



* Packing unit(1box): 50ml

Regeneration whitening & wrinkles improvement functional
Professional Derma Cosmetic

*Clinical trials
- clinically proven product

Contains **NAD⁺**(Nicotinamide Adenine Dinucleotide)
Enhance skin barrier | Skin regeneration
Antioxidant | Anti-aging

Skin repairing High-end total regeneration solution

BrightSilk Serum & Cream is a derma cosmetic line designed for post-treatment care and home care use. It is a regenerative skincare solution that helps revitalize damaged skin and restore it to a hydrated, clear, and healthy complexion.

Application

1. Skin rejuvenation
2. Brightening & Whitening
3. Wrinkle removal
4. Acne scar / repairing damaged skin surface
5. Increasing elasticity, skin tone

How to Use



Step 1.
After cleansing, refine your skin texture with a toner.



Step 2.
Take an appropriate amount of **BrightSilk Serum**, gently apply it to your face, and let it absorb.



Step 3.
Finish by evenly applying **BrightSilk Cream** over the entire face.

Major ingredients

Major raw material PDRN (salmon DNA), activated ingredient works fundamental skin concerns such as regeneration, elasticity, tone, and moisture. The formula also includes powerful ingredients like Hyaluronic Acid, Glutathione, and Tranexamic Acid for brightening and hydration, along with various peptides that promote skin firming and reduce the appearance of wrinkles.

01. Regeneration & Anti-aging
PDRN(Sodium DNA), Adenosine, Madecassoside, Allantoin



02. Brightening & Tone up
Glutathione, Tranexamic Acid, Niacinamide, Arbutin



03. Moisturizing & Hydration
Hyaluronic Acid, Glycerin, Shea Butter



04. Skin elasticity and smoothing & protection
Palmitoyl Tripeptide-1, Palmitoyl Tetrapeptide-7, Madecassoside, Royal Jelly, 12 flower extract, Oriental herbal extract